



Reusable and Antimicrobial Sanitary Pads

Number of Pad

Total Pads: 4 pads per box.

Variety:

3 Day Pads: Designed for daytime use.

1 Night Pad: Longer design for overnight protection.

Price

15,000 TZS per pack. (Retailer)

12,000 TZS per pack. (Wholesale)

Uniqueness and features

- Antimicrobial
- High absorbency
- No irritation
- Reusable
- Comfortable

How to wash

- **Soak:** Place the used pad in cold water with soap for 30 minutes.
- **Rinse:** Rinse thoroughly with clean water until the water runs clear.
- **Dry:** Hang to dry under direct sunlight or in a well-ventilated area.
- **Reuse:** Once fully dried, the pad is ready to be used again.

Important Care Tips:

- Do NOT use bleach.
- Do NOT iron the pads.
- Machine washable using cold water and detergent.



Safepad™ is registered by (TBS) under TZS 1659

Safepad™ is a TMDA-registered Medical Device Class A. No. TAN 25 MDR 0220

Contact Us Now

+255 696 090 909

safepad@atoz.co.tz

Arusha, Tanzania



@safepad.tz



Pedi Inayotumika Tena na Tena na Huzuia Bakteria

Idadi ya Pedi

Jumla ya Pedi: Pedi 4 kwa kila pakiti.

Aina:

Pedi 3 za Mchana: Zimeundwa kwa ajili ya matumizi ya mchana.
Pedi 1 ya Usiku: Ndefu zaidi kwa ajili ya ulinzi wa usiku kucha.

Bei

15,000 TZS kwa kila pakiti. (Rejareja)

12,000 TZS kwa kila pakiti. (Jumla)

Sifa za Kipekee

- Inakinga bakteria.
- Uwezo mkubwa wa kufyonza.
- Haina muwasho.
- Inatumika tena na tena.
- Inakupa uhuru/faraja kamili

Jinsi ya na kufua

- **Loweke:** Weka pedi iliyotumika kwenye maji ya baridi yenye sabuni kwa dakika 30.
- **Suuza:** Suuza vizuri na maji safi mpaka maji yawe masafi/meupe.
- **Anika:** Anika juani au mahali penye hewa ya kutosha.
- **Tumia tena:** Pedi ikiwa imekauka vizuri, iko tayari kutumika tena.

Maelekezo Muhimu ya Tunza:

- USITUMIE dawa ya kuondoa madoa (bleach).
- USIPIGE pasi pedi hizi.
- Unaweza kufua kwa mashine kwa kutumia maji baridi na sabuni.



Safepad™ imethibishwa Ubora na **TBS No. TZS 1659**

Safepad™ imesajiliwa na TMDA kama **Kifaa Tiba Daraja A, No. TAN 25 MDR 0220**

Wasiliana Nasi Sasa

+255 696 090 909
safepad@atoz.co.tz
Arusha, Tanzania



@safepad.tz